

Warning! This symbol indicates important information related to safety. Follow these instructions carefully.



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.

**CE**

2022-05-11

Document number: M10-001EN rev.02

## Intended use

---

Surgeon's step intended to enable elevation of the surgeon or supporting staff for better access during surgery. The product is to be used by medical professionals within a medical facility.




## Safety regulations

---

Maximum load: 150 kg

Before use, ensure the foot surface is not damaged and is clean to prevent slippery surface. Check that the protective plastic plugs are in place and not damaged.

Service and preventive maintenance shall be performed by authorized personnel.

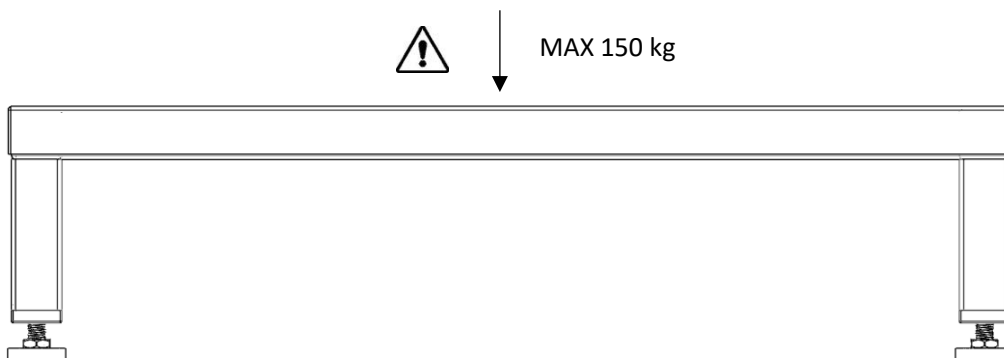

## User instruction

---

### Maintenance

Service, preventive maintenance, adjustment and repair shall be performed by authorized personnel only. Altering the construction without a written consent from the manufacturer, is strictly forbidden.

### Cleaning

Clean with antiseptic detergent with pH 6 – 8 and warm water. Ethanol can be used. Wipe dry with soft cloth. Do not use strong acidic or alkaline detergent.

### Control

Check that there is no damage to any part of the Surgeon's step. Check that the protective plastic plugs are in place and not damaged.

## Technical specification

---

Maximum load	150 kg
Weight	7 kg
Material	Stainless steel
Dimension (H x W x L)	100-200 x 400 x 600 mm (Fixed height specified when ordered)
Adjustable height	20 mm / 100+20 – 200+20
UDI-DI	07350111831721



Reison Medical AB

Eriksbergsvägen 32A

734 92 Hallstahammar

SWEDEN

+46 (0)220 433 99 | [order@reison.se](mailto:order@reison.se) | [info@reison.se](mailto:info@reison.se)