



Article no.	Description	Armboard:	10-380	10-380-K	10-380-F	10-380-L	10-380-M	10-380-M10	10-380-S30
10-181	Plane comfort cushion						x	x	
10-182	Curved comfort cushion						x	x	
10-381	Plane moulded cushion		x		x	x			x
10-381-H	Plane comfort cushion		x		x	x			x
10-382	Curved moulded cushion		x		x	x			x
10-382-H	Curved comfort cushion 45mm		x		x	x			x
10-382-HT	Curved comfort cushion 70mm		x		x	x			x
10-382-K	Curved comfort cushion, short			x					



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.



2020-10-30

Document number: M10-181EN rev.01

Intended use

Cushion intended to be used with armboards for comfort and support to the patient during surgery. The product is to be used by medical professionals within a medical facility.

User instruction

Cleaning

Clean with antiseptic detergent with pH 6 – 8 and warm water. Wipe dry with soft cloth. Do not use strong acidic or alkaline detergent.

Control

Check before use that the cushion is clean and is not damaged.

Technical specification

Version	Description	UDI-DI	Dimension (H x W x L)	Weight	Material
10-181	Cushion	07350111831899	45 x 180 x 610 mm	0,32 kg	Mfoam
10-182	Cushion	07350111831905	65 x 180 x 610 mm	0,33 kg	Mfoam
10-381	Cushion	07350111831936	35 x 165 x 610 mm	0,70 kg	Polyurethane
10-381-H	Cushion	07350111831943	45 x 180 x 610 mm	0,32 kg	Mfoam
10-382	Cushion	07350111831950	25 x 180 x 610 mm	0,80 kg	Polyurethane
10-382-H	Cushion	07350111831967	45 x 180 x 610 mm	0,30 kg	Mfoam
10-382-HT	Cushion	07350111831998	80 x 180 x 610 mm	0,50 kg	Mfoam
10-382-K	Cushion	07350111832018	45 x 180 x 455 mm	0,20 kg	Mfoam



Reison Medical AB
Eriksbergsvägen 32A
734 92 Hallstahammar
SWEDEN

+46 (0)220 433 99 | order@reison.se | info@reison.se