





Radial clamp 10-304 including versions

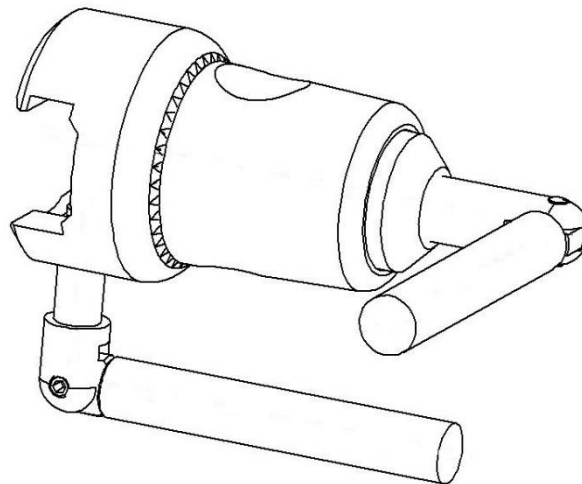

Version	Description	Pole type
10-304	Radial clamp (EU 25x10 mm rail) with V-stud	  Ø 16-20 mm
10-304-P	Radial clamp (EU 25x10 mm rail) with Plane stud	 Ø 16-20 mm



Warning! This symbol indicates important information related to safety. Follow these instructions carefully.



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.



2021-04-19

Dokument nummer: M10-304EN rev.01

Intended use

Clamp intended to attach and position accessories, with a pole size \varnothing 16-20 mm, to the accessory rail on operating and examination tables. The product is to be used by medical professionals within a medical facility.




Safety regulation

Maximum load: 25 kg, according to EN 60601-1 and EN 60601-2.

Before use, ensure correct function!


When loosening the lock levers to adjust the height or clamp position, support the accessory, to prevent it from uncontrolled descend into the bottom position.

Service and preventive maintenance shall be performed by authorized personnel.

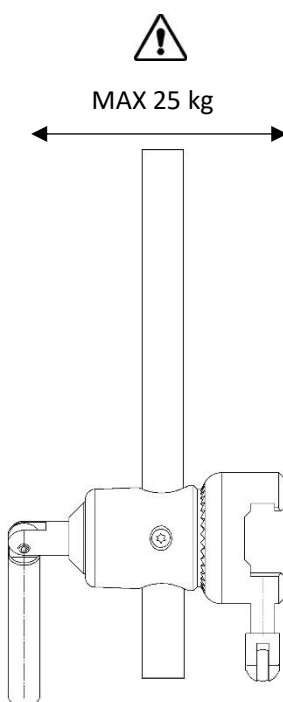


User instruction

Installation

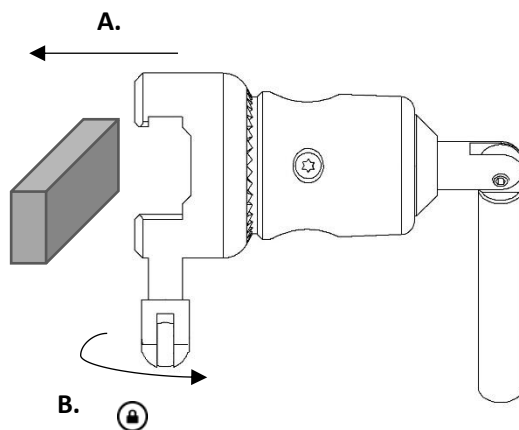

Usage

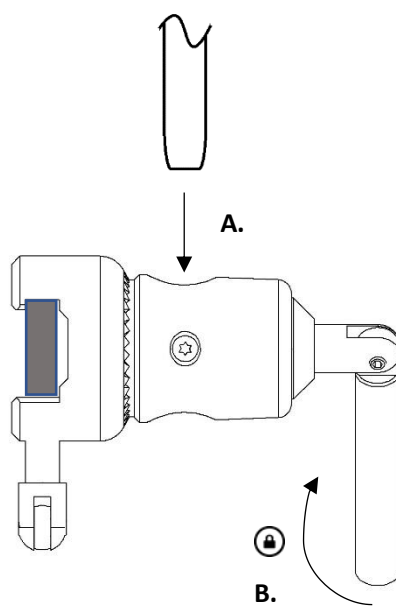

Maintenance



Service, preventive maintenance, adjustment and repair shall be performed by authorized personnel only. Altering the construction without a written consent from the manufacturer, is strictly forbidden.

Cleaning

Clean with antiseptic detergent with pH 6 – 8 and warm water. Ethanol can be used. Wipe dry with soft cloth. Do not use strong acidic or alkaline detergent.


Control

Check that there are no loose parts and that there is no damage to any part. Check the clamp lock function and the function for positioning and locking the accessory pole in the clamp.

Annual service

Once a year the clamp shall be carefully cleaned, moving parts shall be lubricated, the locking function shall be tested

Spare parts

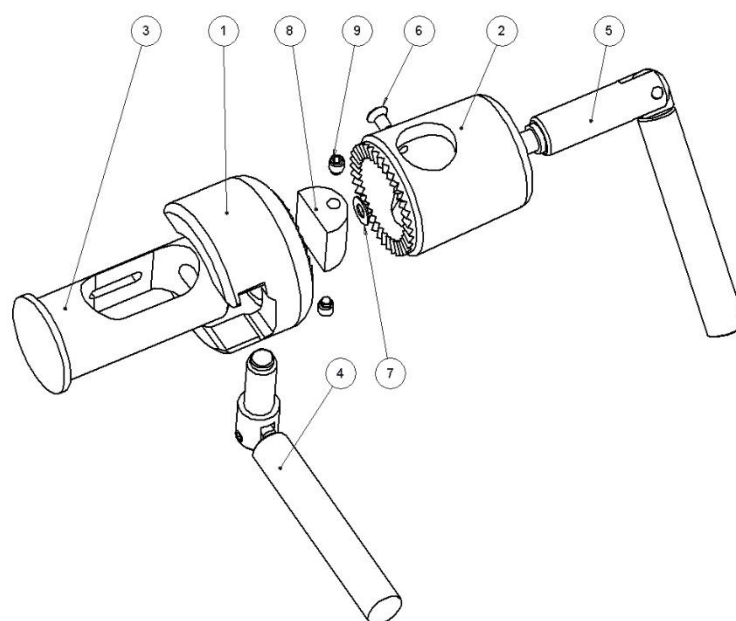

	Description	Version:		
		Pcs.	10-304 Article no.	10-304-P Article no.
1.	Gear ring	1	10315	10315
2.	Gear cylinder	1	10316	10316
3.	Guide	1	10320	10320
4.	Lock handle	1	10062	10062
5.	Lock handle	1	10346	10346
6.	Screw MKFT M4x8 A2	1	-	-
7.	Washer PB 5x10x5	1	-	-
8.	Stud	1	10325	10429
9.	Set screw MT6SS M5x6 A2	2	-	-

Technical specification

Maximum workload 25 kg, according to EN 60601-1 and EN 60601-2.

Weight 1,0 kg

Material clamp
Stainless steel
Brass

Version	UDI-DI	Dimension (H x W x L)
10-304	07350111830052	160 x 55 x 120 mm
10-304-P	07350111830069	160 x 55 x 120 mm



Reison Medical AB

Eriksbergsvägen 32A

734 92 Hallstahammar

SWEDEN

+46 (0)220 433 99 | order@reison.se | info@reison.se