


Head rest 10-333 including versions

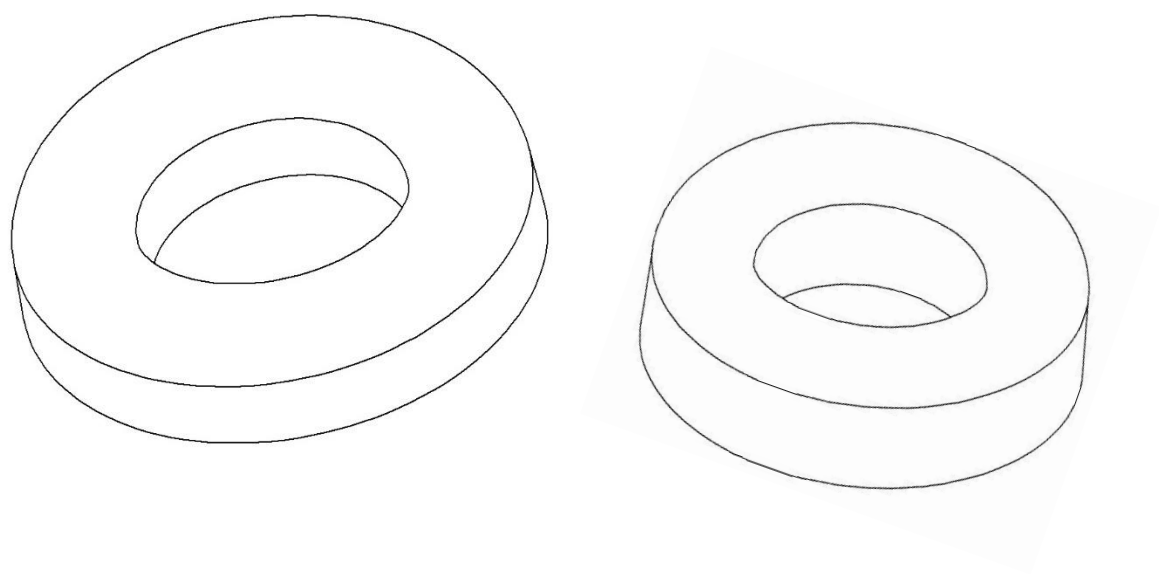

Version	Description
10-333	Head rest for adult patients
10-333-PED	Head rest for pediatric patients



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.



## Intended use

---

Head rest intended to stabilize the patient head during e.g. ENT surgery. The product is to be used by medical professionals within a medical facility.

## User instruction

---

### Usage

Place the head rest on the head plate of the operating table. Then place the patient head in the head rest ring to stabilize the head.

### Cleaning

Clean with antiseptic detergent with pH 6 – 8 and warm water. Wipe dry with soft cloth. Do not use strong acidic or alkaline detergent.

### Control

Check before use that the head rest is clean and is not damaged.

## Technical specification

---

Version	Description	UDI-DI	Dimension	Weight	Material
10-333	Head rest	07350111831325	Outer Ø: 204 mm Inner Ø: 104 mm Height: 50 mm	0,30 kg	Polyurethane
10-333-PED	Head rest	07350111831332	Outer Ø: 148 mm Inner Ø: 77 mm Height: 50 mm	0,13 kg	Polyurethane



Reison Medical AB  
Eriksbergsvägen 32A  
734 92 Hallstahammar  
SWEDEN

+46 (0)220 433 99 | [order@reison.se](mailto:order@reison.se) | [info@reison.se](mailto:info@reison.se)