



Version	Description
10-383	Armboard strap 60 cm
10-383-L	Armboard strap 73 cm



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.

**CE**

2021-04-15

Document number: M10-383EN rev.01

## Intended use

---

Trap intended to be used with armboards to fix the patient arm during surgery. The product is to be used by medical professionals within a medical facility.

## User instruction

---

### Cleaning

Clean with antiseptic detergent with pH 6 – 8 and warm water. Wipe dry with soft cloth. Do not use strong acidic or alkaline detergent.

### Control

Check before use that the strap is clean and is not damaged. Check the velcro locking function

## Technical specification

---

Version	Description	UDI-DI	Dimension (W x L)	Weight	Material
10-383	Strap	07350111832025	50 x 600 mm	34 g	Polyester
10-383-L	Strap	07350111832032	50 x 730 mm	42 g	Polyester



Reison Medical AB  
Eriksbergsvägen 32A  
734 92 Hallstahammar  
SWEDEN

+46 (0)220 433 99 | [order@reison.se](mailto:order@reison.se) | [info@reison.se](mailto:info@reison.se)