

Warning! This symbol indicates important information related to safety. Follow these instructions carefully.



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.

**CE**

2021-03-03

Document number: M10-398EN rev.01

## Intended use

---

Arm surgery table for placement of patient arm during surgical treatment of arm and hand. The product is to be used by medical professionals within a medical facility.



## Safety regulation


---

Maximum load: 32 kg in even distribution, corresponding to a patient weight of 300 kg, according to EN 60601-1 and EN 60601-2.

Before use, ensure correct function!

Remove the arm surgery table during transport!


Service and preventive maintenance shall be performed by authorized personnel.

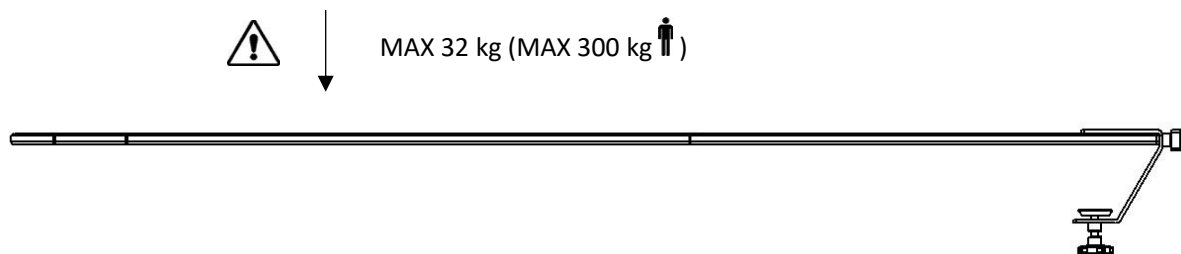

## User instruction

---

### Installation

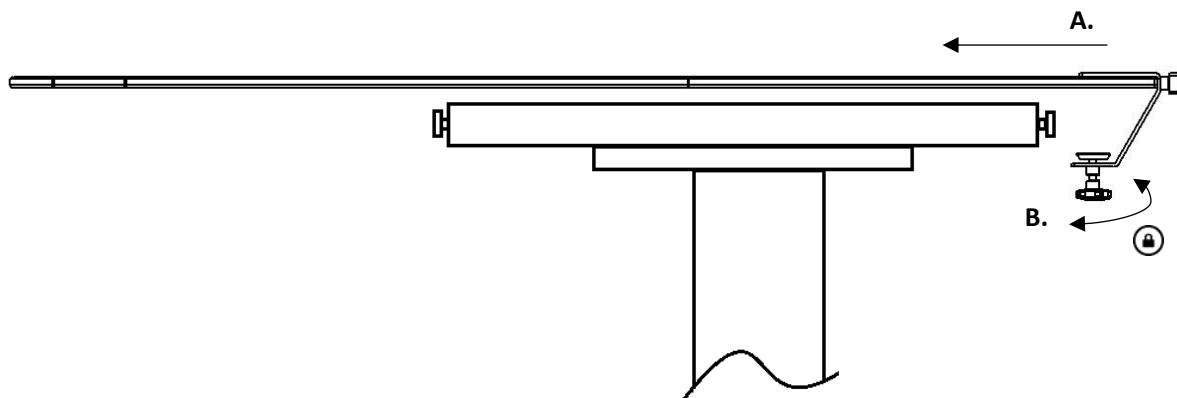

### Maintenance

Service, preventive maintenance, adjustment and repair shall be performed by authorized personnel only. Altering the construction without a written consent from the manufacturer, is strictly forbidden.

### Cleaning

Clean with antiseptic detergent with pH 6 – 8 and warm water. Ethanol can be used. Wipe dry with soft cloth. Do not use strong acidic or alkaline detergent.

### Control


Check that there are no loose parts and that there is no damage to any part. Check the table lock function. If a cushion is used check that it is clean and is not damaged.

### Annual service

Once a year the arm table shall be carefully cleaned, moving parts shall be lubricated, the locking function shall be tested.

## Spare parts

---

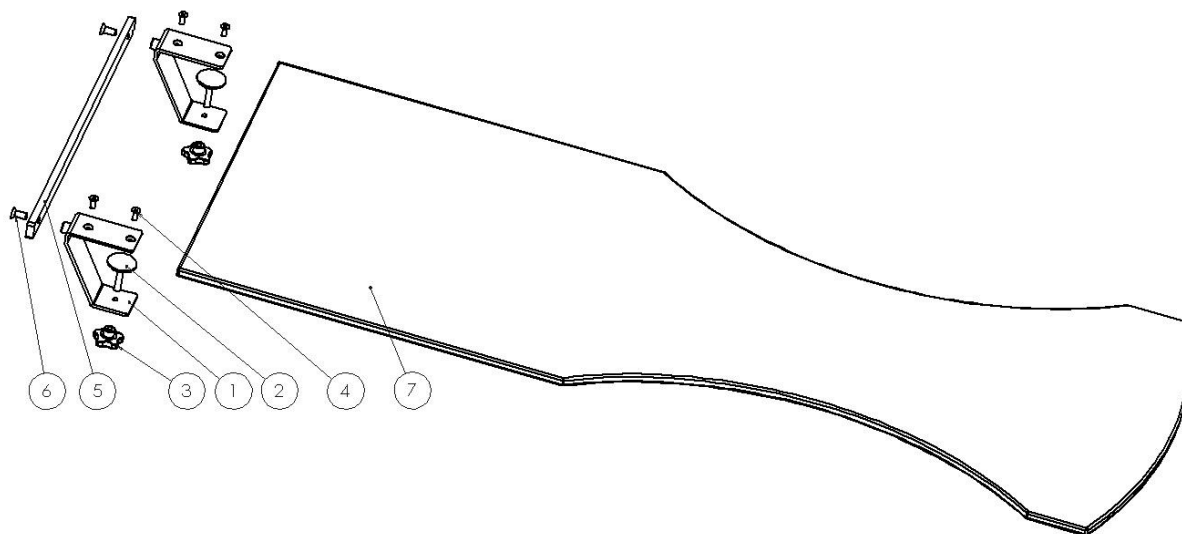

No.	Description	Pcs	Article no.
1.	Bracket	2	11427
2.	Adjustable glide	2	SF3222 M8x70S
3.	Knob	2	VC 192/40 B-M8
4.	Screw MF6S M6x16 A2	4	-
5.	Rail	1	11428
6.	Screw MF6S M8x20 A2	2	-
7.	Plate	1	11384

## Technical specification

---

Maximum work load	32 kg in even distribution, corresponding to a patient weight of 300 kg, according to EN 60601-1 and EN 60601-2
Weight	5,1 kg
Dimension (H x W x L)	140 x 520 x 1370 mm
Thickness plate	13 mm
Material arm surgery table	Carbon fibre Stainless steel
UDI-DI	07350111830892



Reison Medical AB

Eriksbergsvägen 32A

734 92 Hallstahammar

SWEDEN

+46 (0)220 433 99 | [order@reison.se](mailto:order@reison.se) | [info@reison.se](mailto:info@reison.se)