



Version	Description
10-464	Leg strap with padding, hook and velcro
10-464-M	Leg strap with padding, velcro



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.

CE

2021-04-16

Document number: M10-464EN rev.01

Intended use

Leg strap intended to fix the patients legs during surgery. The product is to be used by medical professionals within a medical facility.

User instruction

Usage

10-464: Attach the hook on one of the side rails. Thread the velcro end through the opposite side rail and lock the strap in place with the velcro.

10-464-M: Thread the ends on the strap through each opposite side rail and lock the strap in place with the velcro fastening.

Cleaning

Clean with antiseptic detergent with pH 6 – 8 and warm water. Wipe dry with soft cloth. Do not use strong acidic or alkaline detergent.

Control

Check before use that the leg strap is clean and is not damaged.

Technical specification

Article no.	Description	UDI-DI	Dimension (W x L)	Weight	Material
10-464	Leg strap	07350111832094	125 x 1670 mm	0,3 kg	Polyester
10-464-M	Leg strap	07350111832100	125 x 2410 mm	0,3 kg	Polyester



Reison Medical AB
Eriksbergsvägen 32A
734 92 Hallstahammar
SWEDEN

+46 (0)220 433 99 | order@reison.se | info@reison.se