

Warning! This symbol indicates important information related to safety. Follow these instructions carefully.



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.

CE

2023-03-01

Document number: M10-552/554EN rev.03

Intended use


Arm and leg guard intended to support and keep the patient's arm and leg on the OR-table during surgery and transport. The product is to be used by medical professionals within a medical facility.



Safety regulation

Maximum load: 10 kg in even distribution, according to EN 60601-1 and EN 60601-2.

Before use, ensure correct function!

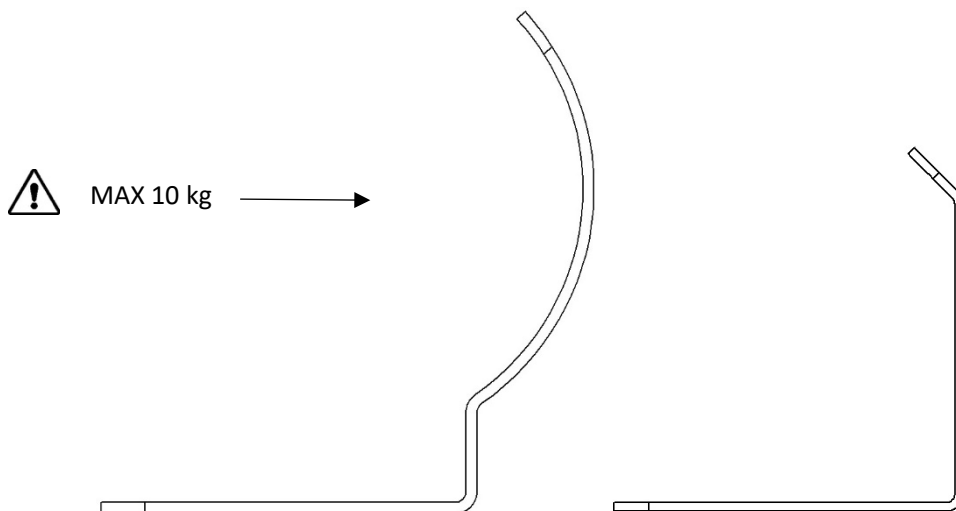

User instruction

Installation

The Arm and leg guard is positioned with the flat end underneath the mattress on the OR-table. The Arm and leg guard is kept in place by the patient weight.

Cleaning

Clean with antiseptic detergent with pH 6 – 8 and warm water. Ethanol can be used. Wipe dry with soft cloth. Do not use strong acidic or alkaline detergent.

Control

Check that there is no damage to any part. If cover is used check that it is clean and is not damaged.

Accessories

Article no.	Description
10-553	Padded cover for 10-552
10-555	Padded cover for 10-554
10-557	Padded cover for 10-556

Technical specification

Article no.	UDI-DI	Maximum load	Dimension (H x W x L)	Weight	Material
10-552	07350111831646	10 kg	230 x 230 x 565 mm	1,5 kg	PETG
10-554	07350111831660	10 kg	210 x 200 x 400 mm	0,9 kg	PETG
10-556	07350111832667	10 kg	230 x 230 x 300 mm	0,8 kg	PETG



Reison Medical AB
Eriksbergsvägen 32A
734 92 Hallstahammar
SWEDEN
+46 (0)220 433 99 | order@reison.se | info@reison.se