

Warning! This symbol indicates important information related to safety. Follow these instructions carefully.



Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.

**CE**

2025-02-14

Document number: M10-448-R EN rev.02

## Latest change

---

Intended use updated to reflect the wording in the Declaration of Conformity.

## Intended use

---

Leg and knee support to hold the leg and/or knee above the operating table during surgery. The product is to be used by medical professionals within a medical facility.

## Description

---

This knee and leg support is designed to keep the patient's legs steady above the operating table. It consists of an angled stand and a swivelling cushion. The support can be raised and lowered to adapt to different patients.



## Safety regulations

---


Before use, ensure correct function!

Service and preventive maintenance shall be performed by authorized personnel.

## User instructions

---

### Installation

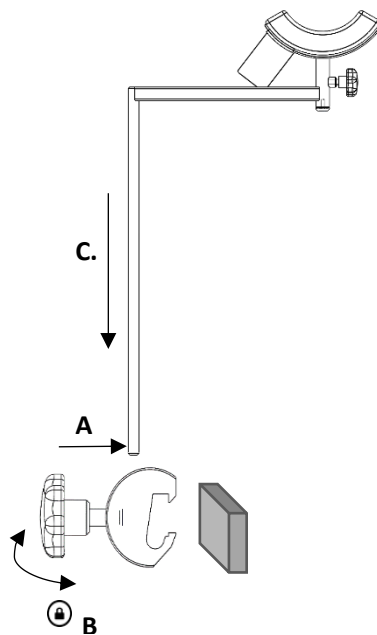

## **Usage**

To attach the knee and foot support to the accessory rail of the operating table, a fixed hook clamp, e.g. 10-301, should be used. The post of the knee and foot support is pushed down into the hole of the clamp after it has been hooked onto the rail. The locking knob that locks the rotation of the cushion must be loosened so that the cushion can be adjusted to the patient's legs. The cushion is then locked by tightening the cushion locking knob clockwise.

## **Maintenance**

Service, preventive maintenance, adjustments and repairs may only be carried out by trained personnel. Modification of any part of the design without the written consent of the manufacturer is strictly prohibited. Violation may be subject to prosecution.

## **Cleaning**

Clean with antiseptic soap detergent with pH 6 - 8 and warm water. Dry with a soft cloth. Do not use detergents containing strong acids or bases.

## **Control**


Check the function and that there is no damage to the knee and foot support.

## **Annual service**

Every year the knee and foot support shall be thoroughly cleaned, see above. Threads should be lubricated with waterproof grease and the locking function of the claw should be checked.

## Spare parts

---

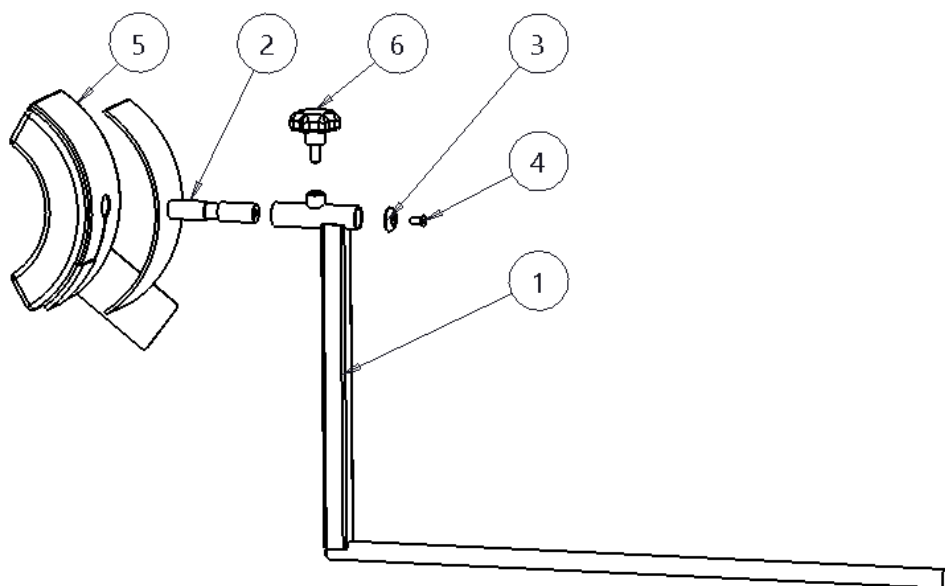

Nr.	Description	Pcs.	Article no.
1.	Stand	1	10507-1
2.	Cushion holder	1	10506
3.	Washer	1	10003
4.	Screw MF6S M5x12 A2	1	-
5.	Cushion	1	10509
6.	Locking knob VCT.43-pM8x16 soft	1	-

## Technical specification

---

Weight	0,7 kg
Dimension (H x B x L)	627 x 85 x 380 mm
Material	Polycarbonate Stainless steel
UDI-DI	07350111832902



Reison Medical AB  
Eriksbergsvägen 32A  
734 92 Hallstahammar  
SWEDEN  
+46 (0)220 433 99 | [order@reison.se](mailto:order@reison.se) | [info@reison.se](mailto:info@reison.se)